

AIR ASPIRATOR NEEDLE

R_x Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Air Aspirator Needle is used to remove entrapped air from the aorta before termination of cardiopulmonary bypass procedures.

Contraindications:

There are no known contraindications specific to this device.



Precautions:

Caution: Inspect sealed sterile package before opening. If seal is broken, contents may not be sterile and may cause infection in the patient.

Caution: For single patient use only. Do not reuse, reprocess or re-sterilize. Reuse, reprocessing or re-sterilization will compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury or illness.

Direction for Use:

1. Attach hub of air Aspirator Needle to air aspiration line.
2. Remove protective cap and insert needle into aorta.
3. Aspirate until all air is removed.



NovoSci[™]

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424002-000 Rev. M

AIR ASPIRATOR NEEDLE

R_x Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Air Aspirator Needle is used to remove entrapped air from the aorta before termination of cardiopulmonary bypass procedures.

Contraindications:

There are no known contraindications specific to this device.



Precautions:

Caution: Inspect sealed sterile package before opening. If seal is broken, contents may not be sterile and may cause infection in the patient.

Caution: For single patient use only. Do not reuse, reprocess or re-sterilize. Reuse, reprocessing or re-sterilization will compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury or illness.

Direction for Use:

1. Attach hub of air Aspirator Needle to air aspiration line.
2. Remove protective cap and insert needle into aorta.
3. Aspirate until all air is removed.



NovoSci[™]

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424002-000 Rev. M