

## VESSEL CANNULA / 1 UNIT

R<sub>x</sub> Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

### Intended Use/Indications:

The Vessel Cannulae are intended to be used during the cardiopulmonary bypass procedure to deliver cardioplegia to the heart.

### Contraindications:

This device is not designed, sold, or intended for except as indicated.



### Warnings and Precautions:

**Warning:** Read instructions carefully. The attending clinician is solely responsible for the set-up and use of this device.

**Warning:** Observe aseptic technique with all tubing connections.

**Warning:** Contents of unopened, undamaged package are sterile, nonpyrogenic. Inspect set and packaging for kinks or damage before use. Sterility and/or product performances may be compromised.

**Warning:** Do not re-sterilize. Single use only.

**Warning:** Do not over-tighten rigid connections.

**Warning:** Dispose of this device according to hospital procedure for contaminated material.

**Caution:** Observe all additional warnings and/or cautions contained in the Directions for Use below.

**Caution:** When infusing cardioplegia, monitor delivery pressure to minimize hemolysis and to prevent damage to vessels.

### Direction for Use:

1. Connect the vessel cannula to the perfusion set.
2. Flush all air from cannula.
3. Place the cannula tip into the vessel to be perfused.

**Caution:** To prevent vessel damage. Monitor delivery pressure during infusions.



**NovoSci**<sup>™</sup>

2021 Airport Rd.  
Conroe, TX 77301  
Toll Free 1-800-854-0567  
Direct 1-281-363-4949

424003-000 Rev. H

## VESSEL CANNULA / 1 UNIT

R<sub>x</sub> Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

### Intended Use/Indications:

The Vessel Cannulae are intended to be used during the cardiopulmonary bypass procedure to deliver cardioplegia to the heart.

### Contraindications:

This device is not designed, sold, or intended for except as indicated.



### Warnings and Precautions:

**Warning:** Read instructions carefully. The attending clinician is solely responsible for the set-up and use of this device.

**Warning:** Observe aseptic technique with all tubing connections.

**Warning:** Contents of unopened, undamaged package are sterile, nonpyrogenic. Inspect set and packaging for kinks or damage before use. Sterility and/or product performances may be compromised.

**Warning:** Do not re-sterilize. Single use only.

**Warning:** Do not over-tighten rigid connections.

**Warning:** Dispose of this device according to hospital procedure for contaminated material.

**Caution:** Observe all additional warnings and/or cautions contained in the Directions for Use below.

**Caution:** When infusing cardioplegia, monitor delivery pressure to minimize hemolysis and to prevent damage to vessels.

### Direction for Use:

4. Connect the vessel cannula to the perfusion set.
5. Flush all air from cannula.
6. Place the cannula tip into the vessel to be perfused.

**Caution:** To prevent vessel damage. Monitor delivery pressure during infusions.



**NovoSci**<sup>™</sup>

2021 Airport Rd.  
Conroe, TX 77301  
Toll Free 1-800-854-0567  
Direct 1-281-363-4949

424003-000 Rev. H