

DISPOSABLE GAS/OXYGEN FILTER

R_X Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Gas Filter may be used to connect the gas source to the oxygenator. It effectively removes bacteria, debris, and other contaminants from oxygen and other medical gases which can cause complications.

Contraindications:

The Gas Filter is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: The Gas Filter is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Aseptically remove the Gas Filter from the pouch.
2. Place filter in the oxygen inflow line of the oxygenator so that the gas flows through the filter in the direction of the arrows on the rim.
3. Connect the gas source to the filter inlet.
4. Make sure the tubing covers the barbs of the filter connectors.



NovoSci[™]

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424305-000 Rev. D

DISPOSABLE GAS/OXYGEN FILTER

R_X Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Gas Filter may be used to connect the gas source to the oxygenator. It effectively removes bacteria, debris, and other contaminants from oxygen and other medical gases which can cause complications.

Contraindications:

The Gas Filter is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: The Gas Filter is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Aseptically remove the Gas Filter from the pouch.
2. Place filter in the oxygen inflow line of the oxygenator so that the gas flows through the filter in the direction of the arrows on the rim.
3. Connect the gas source to the filter inlet.
4. Make sure the tubing covers the barbs of the filter connectors.



NovoSci[™]

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424305-000 Rev. D