

DISPOSABLE “Y” CONNECTOR

R_x Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The “Y” Connector may be used to connect the two caval cannulae to the venous return line and/or to make branch connections in the arterial line during cardiopulmonary bypass.

Contraindications:

The “Y” Connector is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: Always de-bubble the “Y” Connector prior to making patient connections.

Caution: If a luer body is present, ensure that the cap or other device being used is securely on.

Caution: The “Y” Connector is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Select the “Y” Connector with the appropriate branch connector sizes for the intended use.
2. Aseptically remove the “Y” Connector from the peel pouch and transfer it to a sterile field.
3. Push the appropriate connector branches on to the appropriate tubes and cannulae making sure all barbs are covered.
4. Completely de-bubble the “Y” Connector.
5. Proceed with bypass procedure as normal.



NovoSci

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424421-000 Rev. D

DISPOSABLE “Y” CONNECTOR

R_x Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The “Y” Connector may be used to connect the two caval cannulae to the venous return line and/or to make branch connections in the arterial line during cardiopulmonary bypass.

Contraindications:

The “Y” Connector is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: Always de-bubble the “Y” Connector prior to making patient connections.

Caution: If a luer body is present, ensure that the cap or other device being used is securely on.

Caution: The “Y” Connector is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Select the “Y” Connector with the appropriate branch connector sizes for the intended use.
2. Aseptically remove the “Y” Connector from the peel pouch and transfer it to a sterile field.
3. Push the appropriate connector branches on to the appropriate tubes and cannulae making sure all barbs are covered.
3. Completely de-bubble the “Y” Connector.
4. Proceed with bypass procedure as normal.



NovoSci

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424421-000 Rev. D