

DISPOSABLE CONNECTOR REDUCER

Rx Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Connector Reducer may be used to connect two pieces of extracorporeal tubing together in the cardiopulmonary bypass circuit.

Contraindications:

The Connector Reducer is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: Always de-bubble the Connector Reducer prior to making patient connections.

Caution: If a luer body is present, ensure that the cap or other device being used is securely on.

Caution: The Connector Reducer is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Select Connector Reducer with the appropriate sizes for the intended use.
2. Aseptically remove the Connector Reducer from the peel pouch and transfer it to a sterile field.
3. Push the Connector Reducer onto the appropriate making sure all bars are covered.
4. Completely de-bubble the Connector Reducer.
5. Proceed with bypass procedure as normal.



NovoSci

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424423-000 Rev. G

DISPOSABLE CONNECTOR REDUCER

Rx Only

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Intended Use/Indications:

The Connector Reducer may be used to connect two pieces of extracorporeal tubing together in the cardiopulmonary bypass circuit.

Contraindications:

The Connector Reducer is not, in itself, a medical treatment. There are, therefore, no absolute contraindications other than those for the cardiopulmonary bypass in general.



Precautions:

Caution: Always de-bubble the Connector Reducer prior to making patient connections.

Caution: If a luer body is present, ensure that the cap or other device being used is securely on.

Caution: The Connector Reducer is disposable. Do not reuse, reprocess or re-sterilize.

Direction for Use:

1. Select Connector Reducer with the appropriate sizes for the intended use.
2. Aseptically remove the Connector Reducer from the peel pouch and transfer it to a sterile field.
3. Push the Connector Reducer onto the appropriate making sure all bars are covered.
4. Completely de-bubble the Connector Reducer.
5. Proceed with bypass procedure as normal.



NovoSci

2021 Airport Rd.
Conroe, TX 77301
Toll Free 1-800-854-0567
Direct 1-281-363-4949

424423-000 Rev. G